

look inside to  
**SEE WHAT'S  
NEW!**



# READYWRAP™

MANUFACTURED BY SOLARIS

*Medical Binder*

[www.solarismed.com](http://www.solarismed.com)

# READYWRAP

INTRODUCING THE ALL NEW

## All Units

Per your request, Velcro closures are now color-coded to simplify donning!

## Knee Unit

Connects the calf to the thigh for full leg coverage! Available in standard and custom sizes to meet all of your clients' needs!

## Thigh Unit

Now available in standard and custom sizes!

## Calf Unit

NEW design provides greater contouring! Available in standard and custom sizes!

## Foot Unit

NEW simplified design fits into most shoes! NEW left and right options provide better contouring! Standard and custom sizes.



# Table of Contents

- 2 ReadyWrap Full Leg Highlights
- 3 Terms, Pricing, Returns and Warranties
- 3 Contraindications & Materials
- 4 ReadyWrap Products
- 6 ReadyWrap Sizing
- 7 Silver Liners



## Customer Service

**T:** (888) 918-9180  
**F:** (414) 918-9189  
**E:** [info@solarismed.com](mailto:info@solarismed.com)

## Terms, Pricing, Returns, and Warranties

**Terms:** Net 30 days, with credit application approval. FEIN or Reseller identification is required to open a wholesale account. Solaris will add a 1.5% late charge on the unpaid balance of overdue accounts. Solaris will hold orders on wholesale customers with past due accounts.

**Pricing:** Solaris publishes wholesale pricing once a year. Prices are subject to change. Retail prices quoted to consumers reflect a 50% mark-up. Solaris reserves the right to refuse sales to retailers exceeding a 125% mark-up over wholesale pricing.

**Returns:** ReadyWrap products may be returned if found to be defective within two weeks of delivery. Shipping fees are nonrefundable.


**Warranties:** ReadyWrap products are warranted against manufacturer defect for six months.

## Contraindications

- Arterial insufficiency or degeneration
- Deep Vein Thrombosis
- Untreated congestive heart failure, localized or systemic infection, or cancer
- Absent or severely impaired sensation

## Materials

ReadyWrap: Nylon, Lycra, Elastane, Polyurethane, Velcro hook

Silver Liners: Nylon, Acrylic, Polyester, Spandex, 





### **A. RW-LE-AB** **ReadyWrap Foot**

The simple two-strap design fits into most shoes and allows for easier donning. Left and right specific designs enhance contouring. ReadyWrap foot is the perfect alternative for clients who have difficulty donning elastic stockings or whose skin may be compromised by shear forces. Like all of our ReadyWraps, the low stretch material provides a low resting / high working pressure similar to industry standard short-stretch bandaging. Standard sizing accommodates ankle girths up to 38cm, custom designing is also available.

### **B. RW-LE-BD** **ReadyWrap Calf**

Maximum therapeutic benefits, excellent contouring, and complete comfort are the unique features that distinguish the ReadyWrap calf unit. The strapping system mimics the 50% overlap of short-stretch bandaging and reduces shear force pressure when being applied to the limb. Velcro placement is designed to minimize strap shifting and enhance ease of donning. Low-stretch materials provide simultaneous low resting / high working pressure maximizing the calf muscle function, especially when exercising or walking. This unit may be worn alone or over the top of compression hosiery.




### **C. RW-LE-DE** **ReadyWrap Knee**

The ReadyWrap Knee unit provides support for your clients with localized patella swelling, or acts as a bridge connecting the calf and thigh units to provide full leg coverage. Please note that when used to provide full leg coverage, the knee unit should be donned last so the client can easily adjust it when in prolonged sit or stand positions. It's simple, 3-strap design, makes this unit easy to don, and the 50% overlap mimics short-stretch bandaging. Available in standard and custom sizes.

### **D. RW-LE-EG** **ReadyWrap Thigh**

The perfect addition to our knee, calf, and foot units for clients with full leg edema. When used by itself, it is also a great option to prevent localized swelling from the superior patella border to the groin. The strap system's 50% overlap mimics short-stretch bandaging. The low-stretch fabric helps ensure low resting and high working pressure. Our experience has shown that for those clients who have a relatively straight leg, the proximal strap is best placed (straight) across. However, for clients with a cone or V-shaped thigh, having the strap create an X-shape prevents slippage.

# ReadyWrap Standard Sizing

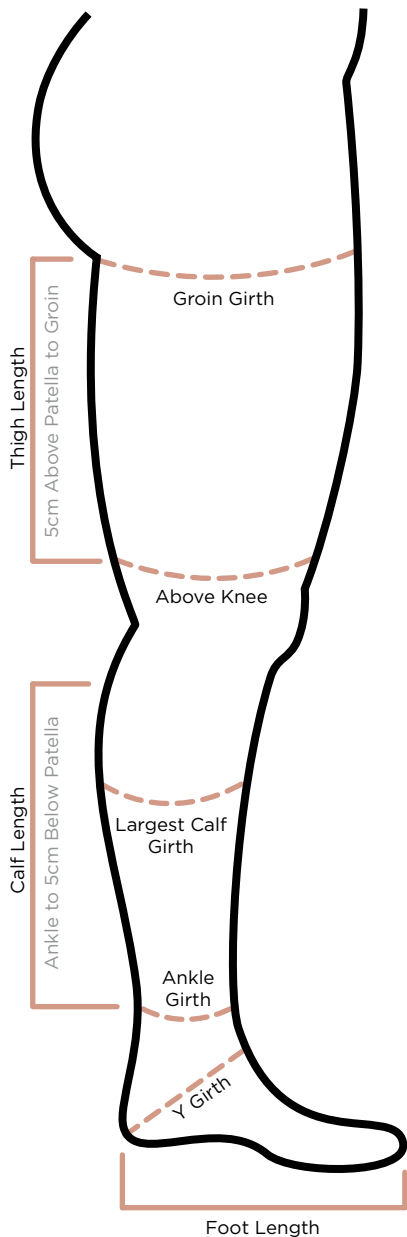

| ReadyWrap Thigh |        |            |             |               |
|-----------------|--------|------------|-------------|---------------|
| Description     | Length | Above Knee | Groin Girth | Item Code     |
| Reg. Small      | 25 cm  | 40 - 55 cm | 50 - 70 cm  | RW-LE-EG-S25  |
| Tall Small      | 30 cm  | 40 - 55 cm | 50 - 70 cm  | RW-LE-EG-S30  |
| Reg. Med.       | 25 cm  | 50 - 65 cm | 60 - 80 cm  | RW-LE-EG-M25  |
| Tall Med.       | 30 cm  | 50 - 65 cm | 60 - 80 cm  | RW-LE-EG-M30  |
| Reg. Large      | 25 cm  | 60 - 75 cm | 70 - 90 cm  | RW-LE-EG-L25  |
| Tall Large      | 30 cm  | 60 - 75 cm | 70 - 90 cm  | RW-LE-EG-L30  |
| Reg. XL         | 25 cm  | 70 - 85 cm | 80 - 100 cm | RW-LE-EG-XL25 |
| Tall XL         | 30 cm  | 70 - 85 cm | 80 - 100 cm | RW-LE-EG-XL30 |

| ReadyWrap Knee |            |            |             |
|----------------|------------|------------|-------------|
| Description    | Calf Girth | Above Knee | Item Code   |
| Small          | 28 - 38 cm | 40 - 55 cm | RW-LE-DE-S  |
| Medium         | 38 - 48 cm | 50 - 65 cm | RW-LE-DE-M  |
| Large          | 43 - 48 cm | 60 - 75 cm | RW-LE-DE-L  |
| Extra Large    | 48 - 68 cm | 70 - 85 cm | RW-LE-DE-XL |

| ReadyWrap Calf |        |              |            |               |
|----------------|--------|--------------|------------|---------------|
| Description    | Length | Ankle Girth  | Calf Girth | Item Code     |
| Reg. Small     | 30 cm  | 18 - 23 cm   | 28 - 38 cm | RW-LE-BD-S30  |
| Tall Small     | 35 cm  | 18 - 23 cm   | 28 - 38 cm | RW-LE-BD-S35  |
| Reg. Med.      | 30 cm  | 20 - 28 cm   | 38 - 43 cm | RW-LE-BD-M30  |
| Tall Med.      | 35 cm  | 20 - 28 cm   | 38 - 43 cm | RW-LE-BD-M35  |
| Reg. Large     | 30 cm  | 25.5 - 33 cm | 43 - 48 cm | RW-LE-BD-L30  |
| Tall Large     | 35 cm  | 25.5 - 33 cm | 43 - 48 cm | RW-LE-BD-L35  |
| Reg. XL        | 30 cm  | 30.5 - 38 cm | 48 - 68 cm | RW-LE-BD-XL30 |
| Tall XL        | 35 cm  | 30.5 - 38 cm | 48 - 68 cm | RW-LE-BD-XL35 |

| ReadyWrap Foot |      |        |              |              |               |
|----------------|------|--------|--------------|--------------|---------------|
| Desc.          | Foot | Length | Y Girth      | Ankle Girth  | Item Code     |
| Small          | L    | 21 cm  | 28 - 33 cm   | 18 - 23 cm   | RW-LE-AB-L-S  |
| Small          | R    | 21 cm  | 28 - 33 cm   | 18 - 23 cm   | RW-LE-AB-R-S  |
| Medium         | L    | 23 cm  | 30 - 38 cm   | 20 - 28 cm   | RW-LE-AB-L-M  |
| Medium         | R    | 23 cm  | 30 - 38 cm   | 20 - 28 cm   | RW-LE-AB-R-M  |
| Large          | L    | 26 cm  | 35.5 - 43 cm | 25.5 - 33 cm | RW-LE-AB-L-L  |
| Large          | R    | 26 cm  | 35.5 - 43 cm | 25.5 - 33 cm | RW-LE-AB-R-L  |
| XL             | L    | 29 cm  | 40.5 - 48 cm | 30.5 - 38 cm | RW-LE-AB-L-XL |
| XL             | R    | 29 cm  | 40.5 - 48 cm | 30.5 - 38 cm | RW-LE-AB-R-XL |

# ReadyWrap Custom Sizing

We know that while most of your clients will fit into our standard sizes, you'll occasionally have one that would benefit more from a custom garment. That's why we're proud to offer custom sizing for all of our ReadyWrap Foot, Calf, Knee, and Thigh units. If you happen to run into a particularly difficult case, our Design Center is always here to help and discuss solutions with you too. Just give them a call at (888) 918-9180 if you have any questions. Custom forms are available in our price lists and order forms catalog or online at: [www.solarismed.com](http://www.solarismed.com) (professional registration required).



**A. LN-LE-AD**  
**Solaris Silver Liner Below Knee**

**B. LN-LE-AG**  
**Solaris Silver Liner Thigh-High**

Solaris is proud to offer a liner specifically designed to be used under compression garments. Every Solaris Silver Liner is manufactured with unique silver, antimicrobial properties for maximum comfort and protection. Solaris Silver Liners are manufactured in the USA and are available in below knee and thigh-high versions. Both versions accommodate girths up to 75cm.

# READYWRAP

MANUFACTURED BY SOLARIS

*Visit us online,  
we welcome all your ideas and feedback!*

**[www.solarismed.com](http://www.solarismed.com)**



Follow us on:



[facebook.com/solarisinc](https://facebook.com/solarisinc)



[twitter.com/solarisinc](https://twitter.com/solarisinc)

Solaris, Inc.  
6737 West Washington Street, Suite 3260  
West Allis, Wisconsin 53214

**Change Service Requested**